

# BIOPTRON 2

PAG-880



## BIOPTRON 2 FEATURES

The BIOPTRON 2 light therapy device is primarily designed for use by **health professionals**. Different stands are available (professional, y-stand, wall-mounted) to assure the best-fitting solution for every need. The easy positioning allows a comfortable treatment for the patient as well as for the medical professional user. On a control panel the duration of each treatment session can easily be preset up to 95 minutes, dividable into minute-steps. The **filter** diameter (approx. 15 cm) permits the treatment of larger areas and hence offers a time effective therapy.

FAST, EASY TO USE AND SAFE

## TECHNICAL DATA

Wavelength  
480 - 3400 nm  
Degree of polarization  
>95% (590 - 1550 nm)  
Specific power density

av. 40 mW/cm<sup>2</sup>  
Light energy per minute  
av. 2.4 J/cm<sup>2</sup>  
Light Intensity  
min. 10.000 lux  
CE labeling



Weight without stand

4.3 kg

Weight with stand


xxxxxx

Power Supply

100-240 V~ 50/60 Hz

Power consumption

1.4-1.0 A

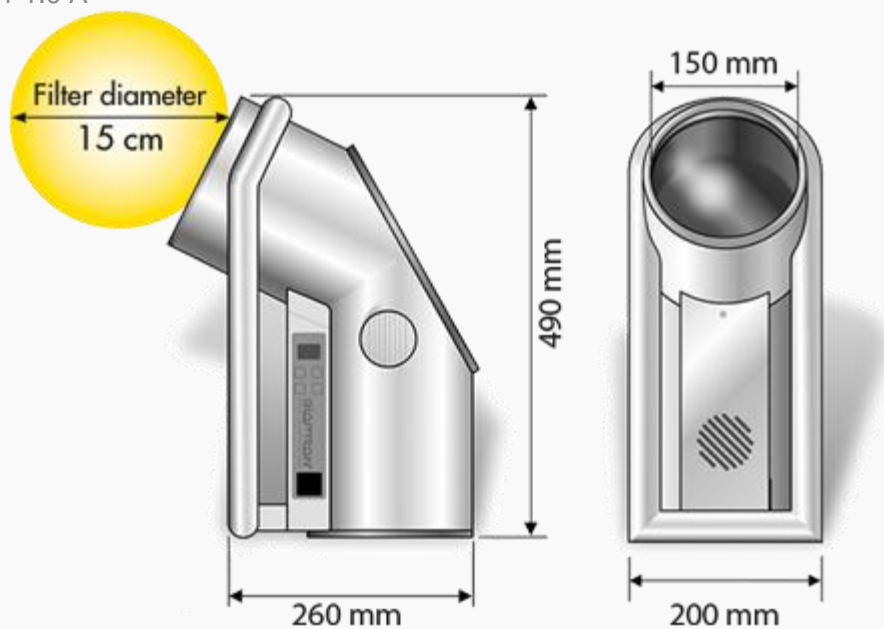

ACCESSORIES FOR BIOPTRON 2



# BIOPTRON 2 ACCESSORIES

## BIOPTRON 2 Domestic Stand (or Y-Stand) (code: PAG-880-SET-2)

BIOPTRON 2 comes with a floor stand that allows optimal positioning and a convenient treatment of the whole body to meet everyone's needs.

### Features

1. Absolutely safe
2. Easy to move
3. Highly comfortable for a relaxing treatment
4. Snap-in security system
5. Flexible as it reaches all body areas
6. Consumer-friendly assembly and adjustment


## HOW TO USE

BIOPTRON Light Therapy is to be performed individually. The **duration** of each session and of the entire course of the treatment depends primarily on your health problem and on the state of the area to be treated. Depending on the medical condition to be treated, the BIOPTRON Light Therapy treatment will only take a few minutes once or twice a day. BIOPTRON should be used when you have time to relax and can enjoy the biostimulative effects of the therapy.

All you need to do is follow these simple steps:



Before treatment, clean the area you wish to treat and remove any clothing or topical medications before the treatment.

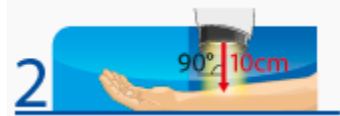

Relax and point the light beam at the area to be treated. Hold the device at an angle of 90° and keep at a distance of approx. 10 cm (4 inches). The light beam should not be moved during the treatment.



Use BIOPTRON once or twice a day e.g. every morning and/or evening for 4 to 10 minutes or follow your doctor's advice.



Repeat the instructions for each area of the body to be treated.



Unplug the Device after use!