



## ActiveBio lamp



Polarized light therapy lamp  
to promote wound healing

### Description

Polarized light emitting lamp that is specifically designed to treat wounds. The filter diameter is approx. 4 cm that allows the selective treatment of small areas, although bigger treatment areas can easily be treated successively if time constraints are not critically important. The outgoing light polarization rate is over 99% . Wavelength range: 400-1000 nm. Powered by AC power supply. Optional accessories available upon order: 1) Colour Therapy Set of 6 further colour lenses: red, blue, orange, green, pink, violet; 2) Stand that allows convenient positioning.

### Recommended for

People who wish to have faster healing skin, less visible scars. It is also recommended for everybody who wants to accelerate the healing process of their wounds.

### Claims

As supplementary treatment the polarized light therapy:

- ✓ Promotes wound healing of different origins: post-surgical wounds, burns, venous ulcer, leg ulcer, pressure sores
- ✓ Relieves pain from inflammation
- ✓ Alleviates acne

### Advantages

Completely drug-free, natural treatment, simple to use. By using the lamp on a stand (optional accessory available upon order) the angle and the position of the lamp can be adjusted to best suit the treatment.

### Instructions for use

The application of the lamp is very simple, no professional skills are needed. Simply switch on the lamp and apply the polarized light therapy over the skin surface to be treated for 4-10 minutes per occasion. For best results apply the treatments on a regular basis, 1-2 times a day. It can be applied at home and in clinical circumstances as well. It is safe, absolutely painless and there are no known side effects.

### How it works

The lamp emits special polarized light in the visible range that leads to photobiostimulation in the skin, and increases microcirculation. Natural light (nonpolarised) vibrates in all directions. In polarized light the light waves move in parallel plains. A polarizer transforms unpolarised light into polarized light by absorbing some of the light and selectively allowing the rest to pass through.

### Packaging contains

Healing lamp with yellow lens, wall adapter, user manual.

Optional accessories available upon order: 1) Colour Therapy Set of 6 further colour lenses: red, blue, orange, green, pink, violet; 2) Stand that allows convenient positioning.