



Saltpipe Original



Salt inhaler to help you breathe more easily and cleanse the respiratory system

Description

Portable ceramic inhaler containing salt crystals from the Transylvanian Praid Salt mine. The salt crystals are placed inside the ceramic inhaler. The active ingredients may last for up to 5 year under normal usage.

Recommended for

People suffering from asthma, respiratory allergies and other respiratory system disorders.

People who need assistance for recovery from respiratory illness, who need an aid to clean their respiratory system and would like to breathe more easily.

Claims

As supplementary treatment the Saltpipe Original can reduce the symptoms of

- ✓ Asthma
- ✓ Respiratory allergies (allergic asthma, allergic rhinitis)
- ✓ Chronic obstructive pulmonary disease (COPD)
- ✓ Residual respiratory problems after quitting smoking

Advantages

A handy, portable product that can be used anytime and anywhere. Curative properties of the Praid salt are well known and appreciated thanks to the existing therapeutic treatment facilities in the abandoned parts of the mine.

Instructions for use

Place mouth over the mouthpiece and breathe in normally through the mouth and exhale through the nose.

Prolonged inhalation is not necessary For best results use regularly for 15-25 mins per day. Suitable for adults and children from 5 years.

How it works

When inhaling, the moisture from the air passing the salt crystals absorbs micron sized salt particles, which then enter the respiratory system with each intake of breath, bringing the beneficial effects of natural salt therapy. The inhaled salt stimulates the natural cleaning mechanisms, which flush away any impurities from the surface of cells lining the lungs

Packaging contains

Saltpipe Original, user manual.